# DKS 130

**Duschklappsitz, wandmontiert**  
Folding shower seat, wall-mounted

ART. NR. / REF: 550 700 100

CE

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- 1. Mounting plates
- 2. Frame
- 3. Seat plate
- 4. Fine adjustable Foot
- 5. Fixing clamp
- 6. Balancing screw

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Dear users,

This instructions helps you and the careers as a basis for sure, safe use of the Folding shower seat and a sensible use of the product possibilities.

The instructions describes the preparation, service, use, servicing and cleaning of the product.

This instructions is to be perused carefully and must be always accessible to all users of the product.

The assembly of the folding shower seat may occur only through the specialised dealers, under use of the subsoil to suitable assembly material.

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Standing insecurity / standing inability with use decrease / of missing functional ability of the lower extremities or other illnesses who are affecting the stability (e.g., strength loss with evil illnesses, chronic, advanced heart circulatory illnesses and lung illnesses) infers with a body position weakness, so that an independent use of the shower is still allowed.

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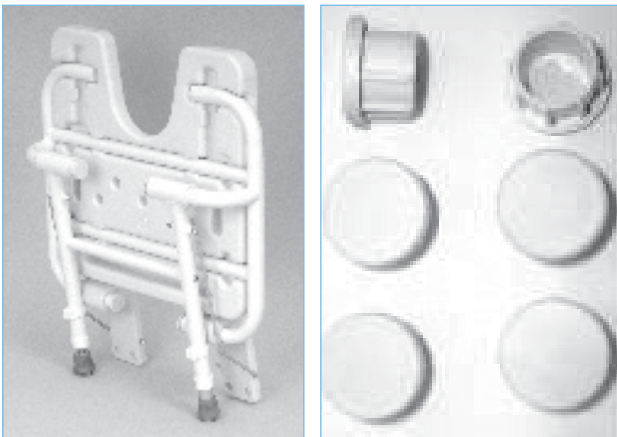
This product is suitable for reapplication. With the processing the standards of the manufacturer must be considered. This standards are made available on inquiry.

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**ATTENTION: The mounting material is not added.** The choice of the connection material is depending on the wall state. Needed are: 8 screws and 8 washers from stainless- steel as well as 8 to the masonry suitable raw plugs.

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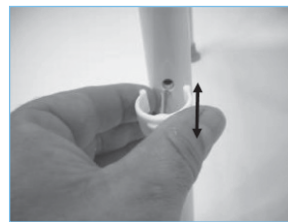
The product contain no dangerous materials. After the product lifetime you supply the single parts according to the material call sign of your local disposal.

The right and entire disposal can be taken over from the manufacturer. This product is suitable for reapplication. With the processing the standards of the manufacturer must be considered. This standards are made available on inquiry.

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To adjust the footrests in the height first unlock the fixing clamp. Then the footrests can be shifted in 4 steps in the height for seat heights from 465, 490, 515 and 540 mm.

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Investigating and adjusting of the seat height must be carried out by an expert.

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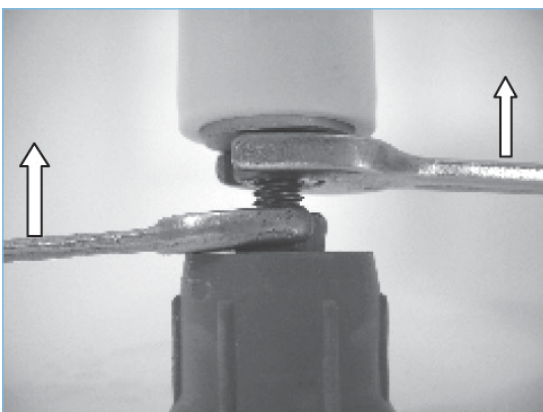
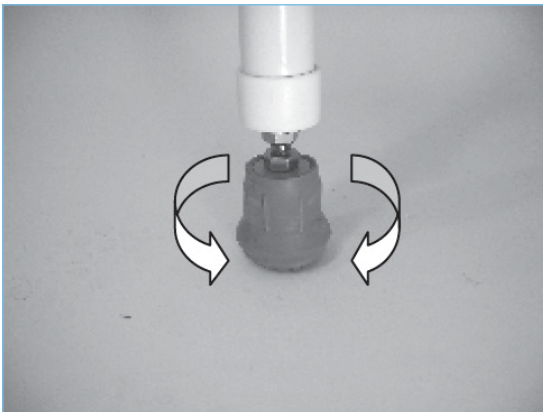
### IMPORTANT !

- Both feet of the user must stand on the ground while sitting.
- Take a normal chair on which the user comfortably can sit.
- Place the user on the chair.
- Measure the height of the seat.
- Choose from the adjustable masses of the Folding shower seat ( 465, 490, 515 or 540 mm ) the measure to the ascertained measure is the next and transfer this measure from the ground of the shower tub on the wall.
- Cut a view window around the centre line from the drilling jig.
- Put the drilling jig on the wall and transfer the 8 drillings. If possible, use the tile seams for drilling so as not to damage the tiles.
- Drill the holes and insert the raw plugs.

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

To compensate possible unevenness's in the shower 1 foot is equipped with a fine adjustment screw (part 6). Adjust the foot at the ground by turning in or out and lock with both nuts as illustrated.

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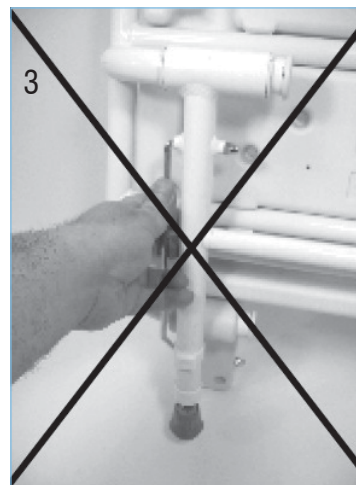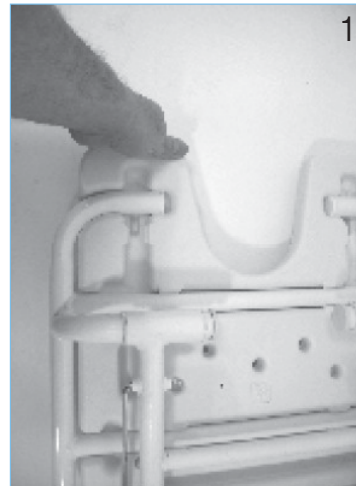To unfold or fold the shower seat when not in use please always use the seatplate as handle (picture 1+2), never do it as shown in picture 3 – risk of finger pinch !

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- 1 Product name
- 2 Article number
- 3 Serial number
- 4 Date of manufacturing year/month
- 5 Attention: See operating instructions

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