

The expected service life of the quad cane is 3 years for a single user if the instructions herein are followed. Failure to follow these instructions can reduce the service life of the device.

warranty details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.



Drive Medical Ltd

Ainley's Industrial Estate, Elland,
West Yorkshire, GB HX5 9JP
www.drivemedical.co.uk

Document code: WS016_Hangtag (Iss 02/12)

quad cane

Manufacturer: Drive DeVilbiss Healthcare
Ltd. • Heathfield Lane • Birkenshaw • West
Yorkshire • BD 112 HW
info@drivedevilbiss.co.uk
www.drivedevilbiss.co.uk
Tel.: + (0) 845 0600 333


Distributor: DeVilbiss Healthcare GmbH
Kamenzer Straße 3 • D-68309 Mannheim
Germany • kontakt@devilbisshc.com
www.drivedevilbiss-int.com
Tel: +49 (0) 621 17898-0
Fax: +49 (0) 621 17898-111



www.drivemedical.co.uk

drive™

about your quad cane

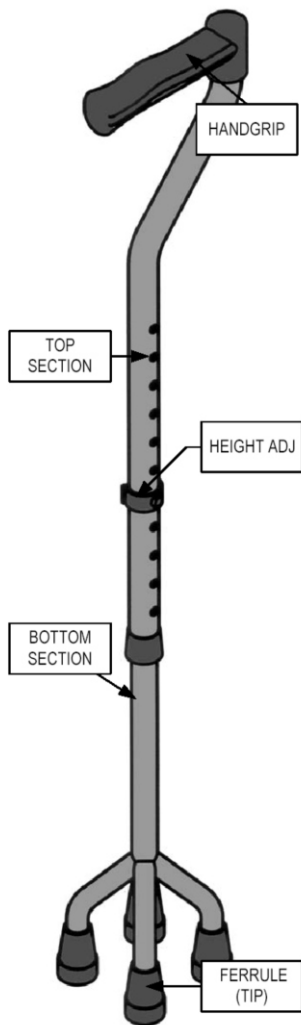

WS016 Standard
Model pictured

Your Drive Quad Cane is height adjustable, lightweight and suitable for indoor and outdoor use. For extra security the bottom section has four legs, each tipped with a grey rubber ferrule.

It is suitable for users up to a user weight of 127kg (20 stone).

Two models of quad cane are available:

The standard model (WS016) has a height of 66 - 92cm (26" - 36").

The larger model (WS016L) has a height of 79 - 104cm (31" - 41") and a larger footprint.

adjusting the height

The ideal height for your quad cane is the distance between the ground and the crease of your wrist. This should allow the user to have an upright posture with their elbow slightly flexed.

For models with 'E' clips

Pull out the 'E' clip to remove. Move the top section until you reach the desired height then reinsert the 'E' clip to secure.

For models with brass buttons

Push in the brass buttons. Extend or retract the leg to the required height. When this is reached, release the button but carry on adjusting until the buttons lock in position.

usage, safety & maintenance

- Avoid wet floors, ice and similar hazards.
- Ensure there is sufficient grip on the surface you are using the cane on.
- Whenever possible use stairs in the normal manner, using the handrail for support. Use the Quad Cane on the same step as the affected leg and lead with the unaffected leg.
- Always ensure the top section is locked in position.
- Periodically check the rubber ferrules have not worn out. See your local Drive Medical dealer for replacement ferrules.
- Periodically check for worn or elongated adjustment holes.
- Always hold the Quad Cane by the handgrip during use.
- Do not exceed the maximum weight limit.