BEN

Badewannendrehstuhl

Swivel Shower Chair

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CE

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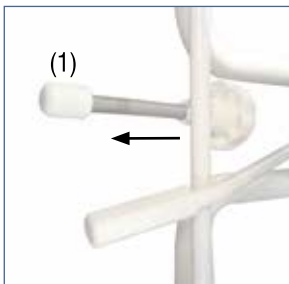
Indication

The swivel bath chair is positioned over the bath and allows the user to swivel in to the bath and sit comfortably whilst they enjoy a bath or shower. They are designed as technical aids to compensate for the users lack of mobility and give the user more independence. The device is suitable only for users who do have some mobility and the required strength to operate the chair and enter and exit the bath using the device.

Assembling


The Swivel Shower Chair is completely assembled for and ready for use.

Fixation in the tub

Turn both setscrews (1) until they touch the bath tub wall to prevent the chair from slipping or moving. Maximum inside width of the bath: 59 cm.

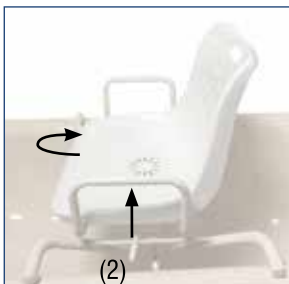Swivel seat operation

The seat can be swivelled after pulling up the stop lever (2). Check this function before use.

Use

- Grip both armrests and slowly sit down on the seat.
- Now pull the stop lever (2) upwards and swing your raised legs in the tub.
- Release the lever and carry on swiveling the seat until it locks in position.
- Repeat the above procedures in reverse to exit the bath and chair.

Safety instructions

- Do not use the Swivel bath Chair if it is damaged!
- There is danger of slipping or falling!

Cleaning

- Do not use abrasive cleaning products.
- Device must not be used when the ambient temperature is over 60°C (e.g. use in a sauna etc.)
- For cleaning and disinfection, use mild household detergents and/or disinfectants.
- Clean by hand with a damp cloth.
- Do not use high-dose chlorine cleaners!

Technical data

Maximum load:	120 kg (19 stone)
Dimensions seat: H/W/D	37 x 46 x 42 cm (14½" x 18" x 16½")
Width over all:	72cm (28")
Height backrest	42cm (16½")
Seat position:	4 steps (0°, 90°, 180° and 270°)
Colour:	white
Weight with backrest:	6,1 kg (13.4lb)
Material frame	Steel, powder coated
Material seat:	high quality plastic (PE-HD)

Warranty

The device has a 24 month warranty from date of purchase.

Type label

1. Product name
2. Article number
3. Serial number
4. Date of manufacturing year/month
5. Attention: See operating instructions

Reuse

Swivel bath chair is suitable for reprocessing after a technical check by a specialized dealer.

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Manufacturer: Drive Medical GmbH & Co. KG • Leutkircher Straße 44 • D-88316 Isny/Allgäu • Germany •
Distributor: DeVilbiss Healthcare GmbH • Kamenzer Straße 3 • D-68309 Mannheim • Germany •